

Using the My-Cardia App for Android

1. Download the free MyCardia app from the Google Play Store and open it.
2. Enter your First/Last Name and Date of Birth and click REGISTER.*
3. Answer "ALLOW" to allow App to use your microphone and to access media files.
4. When you are ready to transmit ECG tap the picture of the myCam device.
5. Place the myCam device 1 inch below the the bottom of your phone with the red button to the right as shown in the app.
6. Press the red button on the myCam to start playing your recording(s).
7. When the recording has completely finished tap the message on your phone to transmit it.**
8. The cardiac technician will call you back within a few minutes. If you do not receive a call - click the "Call My-Cardia for Help" button to be connected.

- * You must use the same name and birth date that your facility entered when you were enrolled.
 ** Do not forget this step or your ECG will not be transmitted!



